



 **Vibringe®**

the sonic syringe

next level
endodontic
hand irrigation.

exclusively from BISCO CANADA.

The Vibringe® is the first endodontic irrigation device to combine manual delivery of the solution with the patented micro-processor controlled sonic technology, which we call the sonic flow technology. This enables delivery and activation of the irrigation solution in one step!

superclean. experience it yourself.




superclean comes standard.




The patented sonic flow technology and specially designed syringes deliver precisely tuned sonic energy to the needle resulting in a thorough debridement and disruption of the smear layer that is more effective than with conventional syringes.




1
Fill up the disposable syringe with your preferred irrigation solution.

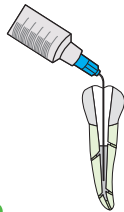


2
Fill the prepared root canal with irrigation solution.

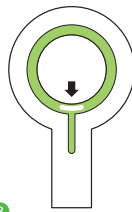


3
Press the on/off activation button to activate the solution.

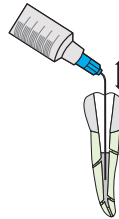

4
Move the Vibringe shortly (30-60 seconds) in 2-3 mm vertical strokes

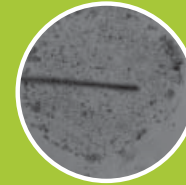


Conventional endodontic hand irrigation

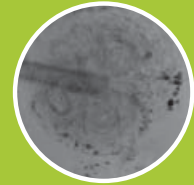

Vibringe sonic flow technology

Hydrodynamic motion

The sonic flow technology of the Vibringe® enables complete irrigation to the apex and allows the solution to reach and disinfect all portions of the canal including lateral canals, fins, webs and anastomoses.

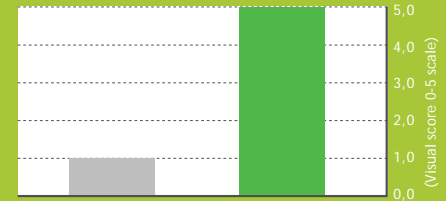

Conventional endodontic hand irrigation Vibringe sonic flow technology

Debris removal

Several studies show that sonic irrigation has better results than conventional irrigation. For more scientific information, visit www.vibringe.com and go to science.



easy.

The Vibringe[®] is in line with your actual irrigation procedure, but will improve your success rate thanks to its patented sonic flow technology which causes acoustic streaming in the root canal. The Vibringe[®] also has an integrated white LED light for enhanced visibility during irrigation.

faster.

Vibringe[®] has clinically proven that it removes organic and inorganic debris from hard-to-reach areas faster than the conventional syringe. The Vibringe[®] combines irrigation & debridement into one easy step. It takes just 30-60 seconds.

safe.

Vibringe[®] is shown to be completely safe for all endodontic irrigation procedures and is compatible with all irrigation solutions including, NaOCL and EDTA. When used correctly it prevents solution and debris from being expressed through the periapical foramen.

superclean.

Vibringe[®] has been clinically proven to enable a more thorough debridement and disruption of the smear layer than conventional hand irrigation. The patented sonic flow technology makes complete irrigation to the apex possible and eliminates blockages. The Vibringe[®] enables the solution to reach and disinfect all portions of the canal including lateral canals, fins, webs and anastomoses. Even during endodontic retreatment the Vibringe[®] can be used for the removal of residual obturation materials.

dentists love it.

The Vibringe[®] was designed and developed in close cooperation with General Practitioners and Endodontists. During this extensive research and development stage, more than 80% of all dental professionals ultimately said they loved working with the Vibringe[®]. It is an easy-to-use device that is in line with conventional irrigation techniques and is more reliable and quicker than other devices!

For more scientific information, visit www.vibringe.com



Vibringe BV is proud to announce that the international jury of the iF Design Award has selected Vibringe for the iF Product Design Award 2009!

International
Forum
Design

superclean. experience it yourself.



- 1 Micro-processor controlled sonic flow technology
- 2 Single-button operation
- 3 Battery charge indicator
- 4 Auto-shutoff with a two-minute timer
- 5 White LED light for user feedback and patient comfort
- 6 Quick connection with 10ml Luer Lock syringe
- 7 Ergonomically designed and lightweight
- 8 Innovative inductive charging system
- 9 Energy efficient

experience Vibringe®. today. call 1-800-667-8811



~~\$844.95~~

Vibringe Sonic Syringe

~~\$129.95~~

***FREE**

Vibringe Disposable Syringes (50pcs)

→ **Introductory Offer**

*Buy 1 Vibringe Sonic Syringe & Receive
A **FREE** Box of Disposable Refill Syringes (50pcs)

*Limited Time Introductory Offer. Mention promo code VIBE01

add the perfect post.

Double Taper LIGHT-POST™ ILLUSION™

The only post designed and recommended by the University of Montreal.

~~\$258.96~~

30 Posts Refill

~~\$92.36~~

10 Posts Refill

~~\$237.96~~

Intro Kit

~~\$48.96~~

Trial Kit

~~\$369.95~~

~~\$131.95~~

~~\$339.95~~

~~\$69.95~~




* Offer only valid with the purchase of the Vibringe Sonic Syringe.

 **Vibringe®**
the sonic syringe

next level endodontic
hand irrigation.



special introductory offer inside.

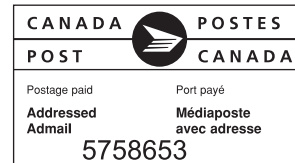

BISCO DENTAL PRODUCTS CANADA INC.
www.biscocanada.com

Head Office: 800-667-8811
info@biscocanada.com

Quebec Office: 800-211-1200
quebec@biscocanada.com